# COVID-19 Testing at Home

## YOUR TESTING KIT CONTAINS:

Nasal swab

Test tube

Buffer liquid

Test cartridge

## YOU WILL NEED:

- A clean, level surface
- Hand sanitizer
- A timer or clock

## BEFORE YOU START


- Sanitize your hands
- Clean surface
- Open the contents of your testing kit
- Remove the test cartridge from the wrapper
- Write your name on the cartridge if testing more than one person

SCAN THIS QR CODE FOR AN INSTRUCTIONAL VIDEO


## STEP 1

- Add 10-12 drops of buffer into test tube until it reaches marked line.
- Insert the swab into your nostril, approximately 2cm.
- Rub and rotate the swab 5 times around and repeat for the other nostril.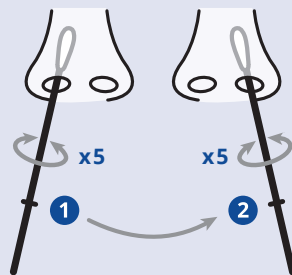


## STEP 2

- Gently tap the test tube to bring the fluid to the bottom.
- Remove blue cap. Immerse swab tip into liquid, swirling up and down for 10 seconds.
- Pinch tube around swab to squeeze out remaining liquid.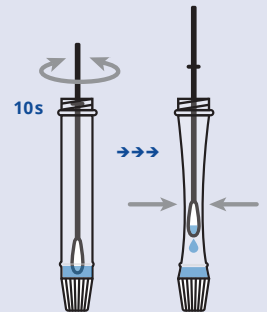


## STEP 3

- Find the break line on the swab handle and snap the swab here.
- Replace the blue cap, with the swab remaining inside of the tube.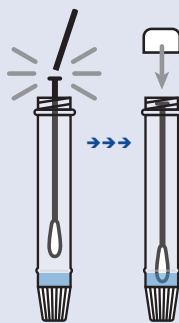

## STEP 4


- Twist off the white dropper cap from the bottom of the test tube.
- Gently squeeze 5 drops into the round well of the test cartridge.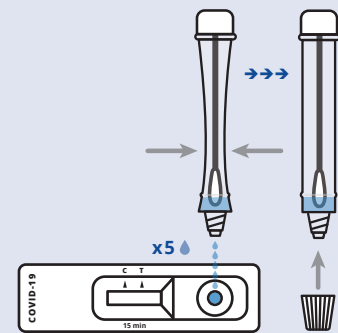

## STEP 5


Let the sample move across the white strip. After a total of 15 minutes, you can determine the result. When finished, put testing supplies in the garbage, sanitize your hands, and clean your testing space.

## NEGATIVE RESULT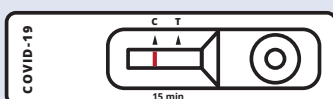

**One line at the C mark, no line at the T mark.** No reporting needed, continue with your testing schedule.

## POSITIVE RESULT

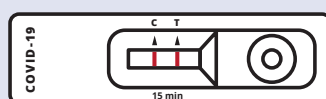**Two lines: one at the C mark, one at the T mark.** You must immediately self isolate. To arrange for confirmation testing, go to [www.gnb.ca/gettestedcovid19](http://www.gnb.ca/gettestedcovid19) or call 811.

## INVALID RESULT

**If your test result does not match either the NEGATIVE or POSITIVE examples, the test may not have worked.** Re-test until you receive a **positive or negative** test result.

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